

Do NOT eat, drink, smoke or chew gum for 30 minutes before collecting the sample.




Fill the tube with saliva to the level of the black wavy line (not including bubbles).

Do not overfill.




Remove the funnel from the tube.

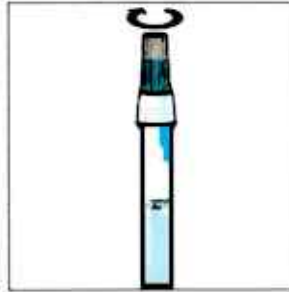

Screw enclosed cap **TIGHTLY** until the blue solution in the cap empties into the tube.



Shake the tube gently for 5 seconds.

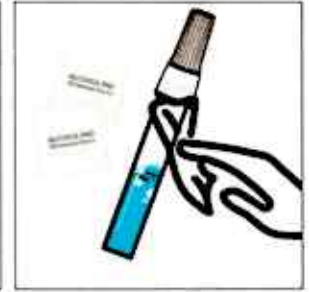

Wipe entire outside of tube and cap with alcohol pad and dispose pad. Place tube in bag.

WARNING: Cap choking hazard, keep out of the reach of children. If at any time the contents of the tube are spilled on your skin, or eyes, wash the affected area with water. **DO NOT INGEST LIQUID**

Saliva Kit Collection Label Guide



There will be 3 barcode stickers on the back of every kit

You will take 1 barcode sticker and put it on any **BLANK** Space of the patient's Requisition



Under each barcode on the collection device there is a small blank space

You will write or label the patients Last Name, First Name and DOB in this space.



PLEASE NOTE: DO NOT COVER ANY PART OF BARCODES

The barcodes help to match a specimen to requisition. If any part of the barcodes are covered this will cause testing delays.